## Vêluses HK – Modèles G121 – G126 – 2060

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### Important instructions for model 2060: Setting the traction power

#### 1. The principle of setting the traction power

You have decided to buy the model 2060 and thus opted in favour of the gentle calving aid. The mechanism of this calving aid has an adjustable traction power control system and allows pulling forces to be applied in two stages, from 50 to 100 daN and from 100 to 160 daN. When the preset traction power has been reached, the handle is locked. This reduces the risk of injuries that can result from unchecked pulling. You should only override this limit in exceptional cases. The traction power measured according to the DLG test report No. 2975 then reaches maximum values of more than 500 daN.

#### 2. Setting the traction power

To set the traction power, the limiting lever (lever with the black top) should be placed in the desired position. Initial settings for the pulling aid should not exceed 100 daN. Once this traction power has been reached, the handle is locked. You should then wait for the animal's next contraction. If the handle continues to be locked despite the next contraction having started, the traction power is too low. The next-higher setting should be chosen, using the limiting lever. To do this, use your other hand to push down the pin next to the number 100. The setting up to 160 daN is sufficient to successfully complete the great majority of births. It may be necessary, in a few individual cases, to override the traction power limit. To do so, press down the pin in the red triangle and at the same time move the limiting lever.

**Warning:** When using the maximum possible traction power, there is a high risk of injury to the mother and calf. We suggest that you consult a veterinary surgeon before overriding the traction power limit.

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# HK-GEBURTSHELFER

## Modell G121 - G126 - 2060

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### HK calving aid Operating instructions

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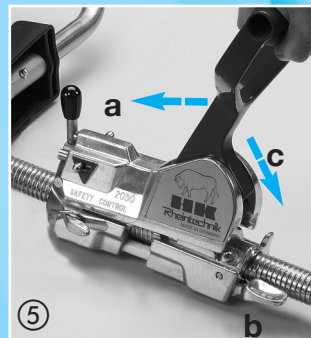
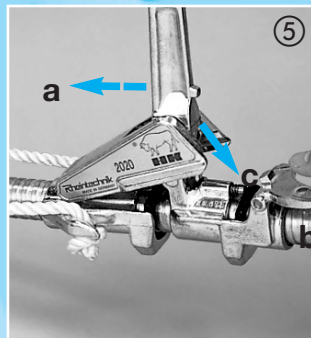
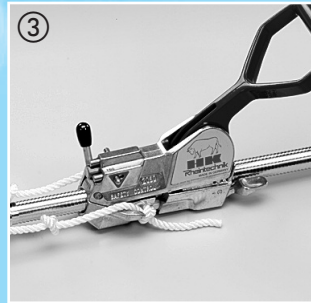
- ① Fixing the ropes
- ② Fixing the attachment support
- ③ Hanging on the ropes
- ④ Pulling during the process of birth
- ⑤ Quick release mechanism 2020/2060

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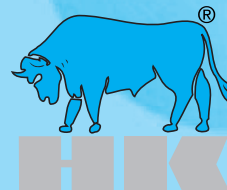
## HK Calving Aid – Model G121 – G126 – 2060

### Fundamental points regarding the calving aid

Applying mechanical pulling aid requires a sound knowledge of the process of birth. Be patient and do not interfere too early. If the birth takes a normal course assistance is necessary only 1 to 2 hours after bursting of the amniotic sac (waterbag). Mechanical pressure is reasonable only if, for head presentation, the two front limbs and the head of the calf are stretched out and if, for breech presentation, the rear limbs and the pelvis are stretched out and have enough space in the pelvis of the mother. For other presentations and presentations that cannot be corrected as well as in case of an oversized foetus a vet should be consulted. In each situation you should apply pressure carefully, that means pull in small doses in accordance with the contractions and pay attention to pull in the correct direction (see pos. 6). Otherwise you run a severe risk of hurting the mother or the calf.

### After installation of your HK Calving Aid according to the instructions proceed as follows:

1. Arrange a clean place to lie the new born calf preferable with straw. Also take care that you have enough space for fixing your HK Calving Aid. Clean the bearing of the animal with a neutral soap and rinse with clean water.
2. Thoroughly examine the cow to get an exact overview of the position of the calf. Pay attention to work with utmost cleanliness. Wash your hands and arms and apply a sterile lubricant before carrying out any correcting examination.
3. Thoroughly clean the calving ropes before each application. Make a loop, place it around the pasterns (Fig. 1) and tighten it.
4. Arrange the attachment support of the HK Calving Aid crosswise directly under the orifice of the vagina (Fig. 2). If you use the support yoke (Model G126 and 2060) adjust it according to the pelvis width to make sure it does not slip to the bottom. For adjustment of the support yoke press the spring studs respectively, push to the required width and let them engage in the holes provided. When having the HK-Flexi clamp, either small sized for milk breed (52 cm) or big sized for meat breed (57 cm), push it according to the pelvis in a way that the movable part of the clamp lies directly under the orifice of the vagina. Caused by that all kinds of slipping will be avoided.
5. Suspend the calving ropes at the hooks of the clamping mechanism via the prepared knots (Fig. 3). It should be as short as possible. Choose the hooks that are arranged at the same height opposite each other if you want to pull both limbs at the same time. Alternatively if you prefer to pull one limb at a time attached to the hooks that are arranged opposite each other. The ratchet mechanism 2060 has only two hooks for pulling simultaneously at both limbs.

Please note: According to the latest scientific knowledge pulling should be effected simultaneously at both limbs. Careful working makes the passage of the shoulders and the pelvis easier. Moreover, application of the entire pulling strength on one hoof could possibly cause injuries and should be avoided.

6. While observing the contractions move the handle of the ratchet mechanism up and down and assist by pulling in a controlled manner. If possible, take care that the ropes / chains are always stretched. The correct pulling direction is very important to achieve the desired success of the mechanical pulling aid. In the beginning pull straight in accordance with the extension of the spinal column of the mother (see direction of the arrow A on Fig. 4). When the head and the shoulders (or the pelvis in case of breech presentation) have come out pull towards the udder by slowly tilting the pulling bar (see direction of the arrow B on Fig. 4) downwards.
7. **Quick release system of mechanism 2020/2060.** You have made up your mind for the advantages of the quick release system the handling of which is very easy. Press down the release lever (Fig. 5 b) with your thumb. To reach the starting position of the ratchet mechanism the release lever must be pressed down. With your second hand simultaneously pull the handle downwards to the reach locking plate and hold it in this position. The ratchet can now easily be pushed in the direction of the attachment support. Please note: In case of an extreme pulling load we recommend to push the handle of the ratchet upwards in the direction of the attachment support first (see direction of the arrow 'a' on Fig. 5). Thus the rear chamber is relieved. After that push the release lever (see Fig. 5 'b') and push the handle downwards to reach the locking plate (see direction of the arrow 'c' in Fig. 5). Hold it in this position and move the mechanism to the starting position.

### 8. Cleaning and care

Thoroughly clean the equipment after each application. Grease the pressure springs and bolts now and again. To make possible unscrewing of the Calving Aid for transportation without problems you should also grease the intermediate piece - thread (see item 6 of the spare parts list) some time or other. According to their use the top and bottom parts of the rods and locking plates (see item 2, 3 and 10 of the spare parts list) are subject to natural wear and tear.