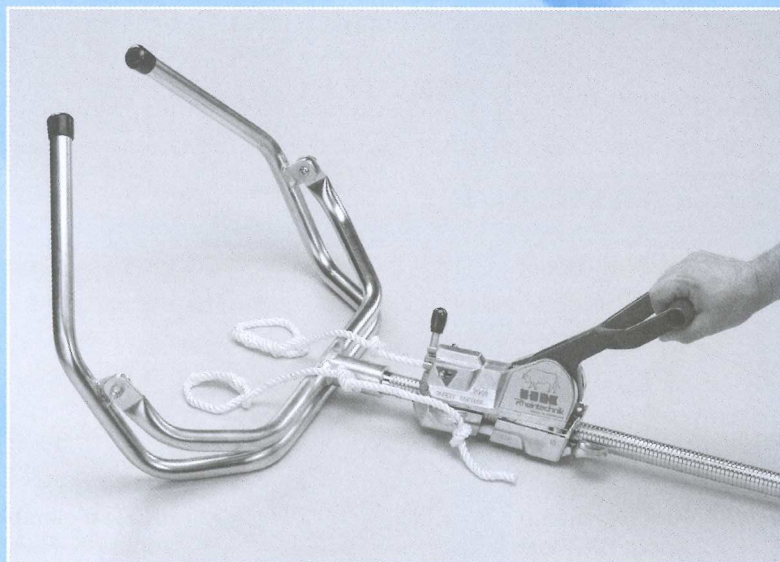
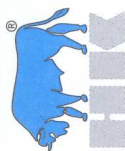
HK-GEBURTSHELFER

Modell G121 - G126 - 2060

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
HK calving aid
Assembly instructions

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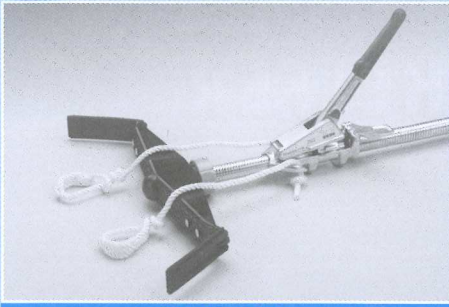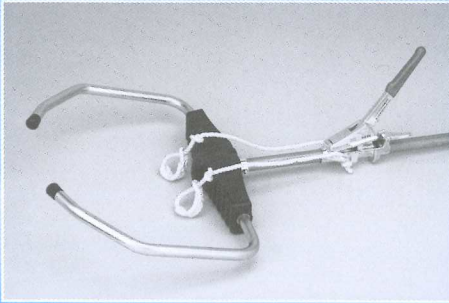
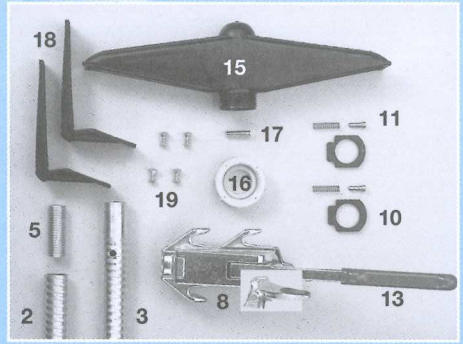
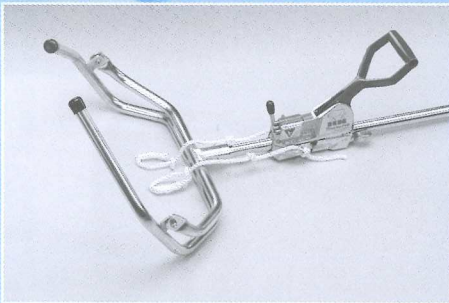
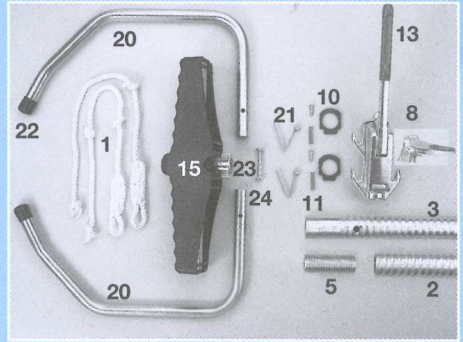
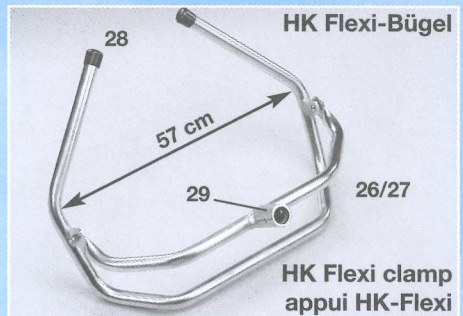
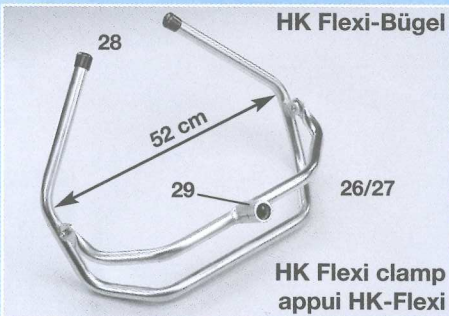
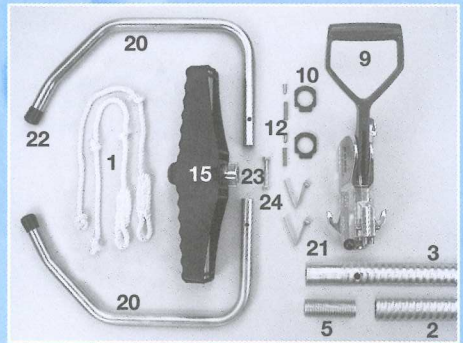
1. General information.

Each HK-calving aid is partially preassembled. It consists of an upper part with a bore for taking up the head piece and a lower tube part with a cap. A threaded piece is enclosed to screw the bars together. The ratchet mechanism 2020 is always mounted on the upper part of the tube. The mechanism 2060 is either mounted on the upper part of the tube or, when having the model with the HK-Flexi clamp, just enclosed. According to the model, the following parts are included: a small head piece with angled corner pieces or a big head piece with a bow, or, in a separate box, the HK-Flexi clamp, either small sized or milk breed or big sized for meat breed. Ropes in pairs, operating and mounting instructions and a list of spare parts are enclosed with each unit.

2. Assembly of G121 model.

Screw the upper (3) and lower (2) part of the tube together using the threaded piece (5). Insert the grey union nut (16) with the small opening first on the upper part of the tube. Attach the head piece (15) and twist until the hole lies directly above the hole in the upper part of the tube. Place the bolt (17) into the hole and screw the grey union nut (16) tightly on to the head piece. Mount the angled pieces (18) with screws and nuts (19) in each case on the left and the right of the head piece.

3. Assembly of G 126 and 2060 model.

Screw the upper (3) and the lower (2) part of the tube together using the threaded piece (5). Attach the head piece (15) to the upper (3) part of the tube and twist until the hole of the aluminium link (23) lies directly above the hole of the upper part (3) of the tube. Push the screw M8 x 55 (24) through the aluminium link (23) and the upper part of the tube (3) and fix with nut. Stick the bows (20) into the lateral opening of the head piece. Push in the spring head, telescope the bows to the desired dimension and let it lock in the bores provided.

4. Assembly of G126 and G2060 model with HK-Flexi clamp.

When having the ratchet mechanism 2060 (9) with the HK-Flexi clamp, the ratchet has to be mounted on the upper part of the tube. Therefore, the release lever (see operating instructions Fig. 5 b) must be pressed and held in this position. After that, attach the HK-Flexi clamp normal sized (26) or big sized (27) to the upper part of the tube (3), twist until the hole lies directly above the hole of the tube and fix with screw M8 x 50 (29) and nut.

5. After assembly.

Pull the ratchet mechanism over the upper and the lower part of the tube by moving the handle up and down. Reset the ratchet mechanism to its original position by releasing the locking device (see operating instructions).

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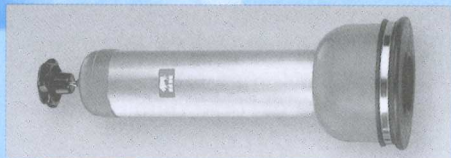
HK pregnancy diagnosis unit for cows

Proved quality ... new technology and design.

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HK-calf resuscitator

Cause the first minutes will be decisive. To take mucus out off calves nose and mouth and for stimulation.

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HK-birth alarm system

This electronic measuring and alarm systems gives reliability, rest and peace of mind, by day and night. Unlimited transmission range and mobility by telephone-alarm!

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An ultrasound inhaler sets the new pattern for treatment of the horses respiratory tracts

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GmbH & Co. KG