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How to use

Note:

Always follow the care instructions. This is the only way to permanently ensure the smooth operation of the syringe.

1. Set the desired dosage using the black set screw (on the handle). You can see the set dose on the cylinder scale (front edge of the plunger).
2. Push the plunger all the way in (up to the stop on the front side of the cylinder) and immerse the tip of the nozzle into the liquid to be administered.
3. Release the handle and the integrated spring will pull in the liquid to be administered up to the pre-set dosage level (see 1).
4. The syringe is now ready for use.

Care instructions

Care after use

CAUTION!

Risk of material damage if used incorrectly.

Do not use any acidic or solvent-based cleaning agents.

Do not boil the syringe or the individual components, this could damage the plastic and/or reduce the service life of the product.

After use, remove any residue with conventional cleaning agents/detergents and rinse the syringe thoroughly with hot water.

Oiling the sealing ring

(if the syringe is sticking)

CAUTION!

Risk of material damage if used incorrectly.

Do not use any mineral oils, silicone oils, WD-40, glycerin or alcohol.

Petroleum-based oils could also damage the sealing ring and/or reduce the service life of the product.

1. Unscrew the cylinder from the black mount (hold the mount firmly) and remove it from the plunger.
2. Apply 100 % extra virgin olive oil, cold pressed, or equivalent cooking oil to the sealing ring. Please note the manufacturer's shelf life and storage instructions.
3. Reattach the cylinder to the plunger and screw it into the black mount.

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